## What is Earwax?

Earwax is a naturally occurring substance that lubricates and protects the inside of your ear from dirt, bacteria, and anything else that might try to sneak its way into the ear canal. However, when earwax builds up and creates impaction, negative side effects can occur.

## Impacted Earwax – it's not about hygiene

Earwax is nothing to be embarrassed about, more than 18 million people a year see their doctor for earwax, so don't think you are alone. It is natural and very common.

Although it's tempting to use a cotton swab, bobby pin or even keys to scrape out unwanted wax, these methods are extremely dangerous, as is ear candling. These methods should be avoided at all costs.

## Are you suffering from earwax impaction?

Symptoms include:

- Hearing loss
- Fullness in the ear
- Itching
- Tinnitus

## Hearing aids and ear buds often increase the production of earwax

If you wear hearing aids, it is important to clean your hearing aids every day so that wax does not build up on the device.

## What are your options for removing impacted earwax?

Until now, over-the-counter earwax removal aids have only been mildly effective in removing earwax. Manual extraction by a doctor is often the go-to method to remove impacted wax. The extraction is often uncomfortable and time consuming.

That is why we developed Earwax MD.

**Avoid these types of harmful earwax removal techniques! They are dangerous and can lead to serious damages to the eardrum and ear canal.**

- Cotton swabs
- Bobby pins
- Car keys
- Ear candles

## How to Use Earwax MD in 6 Easy Steps:

### Step 1

Lie down on your side in a comfortable position. Be sure your ear is parallel to the surface so that nothing drips out of your ear. Place a towel on your shoulder just in case anything drips.

### Step 2

Take the bottle of Earwax MD and fill the dropper with product.

### Step 3

With your head tilted to the side, slowly fill the ear canal with Earwax MD, one drop at a time. All ear canals are shaped differently, so the amount of drops required to fill the ear canal will vary. Once your ear canal feels full, you are finished with the drops.

### Step 4

Set a timer for 15 minutes and wait.

### Step 5

After 15 minutes, it's time to rinse your ear. There are several ways to rinse your ear:

- Fill the purple bulb with warm water and lean your head over a bowl or bathroom sink to flush the ear canal
- Stand under the shower head with your head tilted to let warm water stream into ear canal and flush out the wax
- Use Eosera's WaxBlaster MD to spray a warm water rinse and lean your head over a bowl or bathroom sink to flush the ear canal

You may see cloudy water, or wax coming out in chunks. This means it's working!

### Step 6

Repeat steps 1-5 on your other ear!

## Frequently Asked Questions (FAQs):

### How do I know if my ear is clean?

After using Earwax MD, if you notice your ear feels more clear, or your hearing has improved, chances are your ear is clean! However, after using Earwax MD more consistently, your ear will gradually become cleaner and clearer over time.

### What do I do if Earwax MD doesn't work for me?

If Earwax MD does not work after one 15-30-minute treatment, repeat treatment over the next few days. If desired results are not reached, you may need to consult your doctor. Following a professional ear cleaning, use Earwax MD at-home for routine maintenance. Ask your doctor how often you should use Earwax MD (i.e.: twice per month, once per month, once every 3 months, etc.).

### How can I contact customer service?

For the most efficient customer service, call us at 844-7EARWAX. Visit our website at [www.earwaxmd.com](http://www.earwaxmd.com) and complete the contact form.

### Stay connected with us for tips on ear cleaning and overall ear health!

Subscribe to our blog on [www.earwaxmd.com](http://www.earwaxmd.com), and follow us on Facebook, Twitter or Instagram (@EoseraInc) for special offers and promotions. Send us pictures with the product, or using the product, to [info@eosera.com](mailto:info@eosera.com) to be featured on our social media pages.

### NOTE:

If unsatisfied with your first 15-minute treatment, you may need an additional 15-minute treatment in each ear. In clinical studies, 86% of patients treated with Earwax MD had a complete clearance of earwax in 15-30 minutes. However, all earwax is different, and a stronger rinse may be needed. If using Earwax MD at-home is unsuccessful, please consult your Audiologist or ENT and ask for assistance cleaning your ear. After professional cleaning, Earwax MD may be used for routine maintenance in the comfort of your own home in order to prevent severe wax buildup. Ask your doctor for a recommendation on how often to use Earwax MD to avoid harmful earwax impaction.

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